



MY SAFE PASS™

Olfactory Test Kit

REPORTING

Symptom screening test for
Neurological Disorders, Concussion TBI, Viruses,
and other neurological diseases that cause olfactory
disorders.

POWERED by: My Safe Pass™

Reporting: Overview

Administering the MSP test or allowing your patient to self administer the test permits the medical provider to receive the results upon test completion.

To begin the test, the administrator uses an office tablet, designated cell-phone or scanner to scan the QR Code in section A to access the MSP testing app.



Reporting: Overview

Next, enter username and choose medical provider(Providers names are added to drop down menu). The patient/administrator checks the term and conditions, allowing MSP to store patient results and gives permission to provide results only to the administrator chosen. Then, scan the QR Code in section B to start the test.

MY SAFE PASS[™]
Olfactory Symptom Screening

YOUR PASS TO FREEDOM

FIRST NAME *

Jeffrey

LAST NAME *

Schwartz

Was this test issued to you by a medical provider? *

Yes No

ENTER

Enter Patients Name

MY SAFE PASS[™]
Olfactory Symptom Screening

Your Profile:

Jeffrey Schwartz

Select Organization / Healthcare Provider

Choose the medical provider or organization that provided you with this test:

Search to Select

Note: Test results are forwarded to the provider selected.

Next

Enter Provider

MY SAFE PASS[™]
MSP

INSTRUCTIONS:

A. Scan QR Code A to access the My Safe Pass[™] app or visit: app.mysafepass.us

B. To start your test, scan QR Code B using the My Safe Pass[™] app on your device.

C. Using the My Safe Pass[™] Olfactory Scent Card, rub Scent 1 with your finger.

D. Smell Scent 1 on the Olfactory Scent Card after rubbing.

E. On the My Safe Pass[™] app, select the scent image that best represents the smell of Scent 1.

F. Select the intensity level of the selected scent.

G. Repeat steps C - F for all remaining scents in numerical order (2 - 5).

H. Answer questionnaire and get your result!

Scan B) QR Code

Reporting: Overview

Start the test and receive results.



Scan Card



Rub Card



Smell




Select




Pass/Fail

Reporting: Overview

Once the test is completed, the provider receives a pass/fail and full patient and provider detail. No need to wait for results, and no need to retrieve patient data. All testing information is provided to help you better assess the patient. In addition, results and proof of testing is emailed to your billing department, saving your office time and cost. As an added benefit, administrators can obtain full use of our software for real time monitoring and compliance. As an option, providers may choose to allow patients to self administer the test in the lobby while in the waiting room, using their own device. They simply follow instructions, and after completion, the results are instantly sent to your office email address of choice.

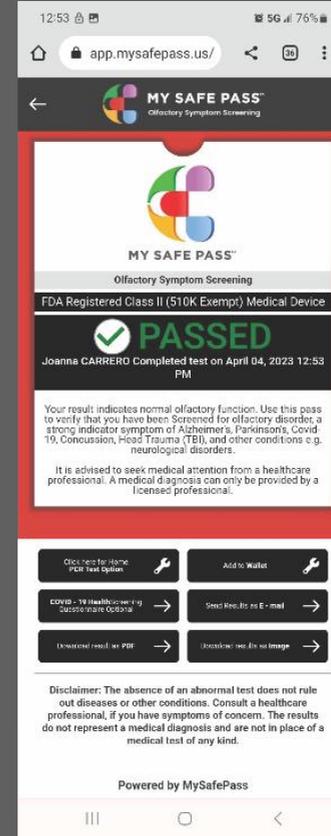

Device View

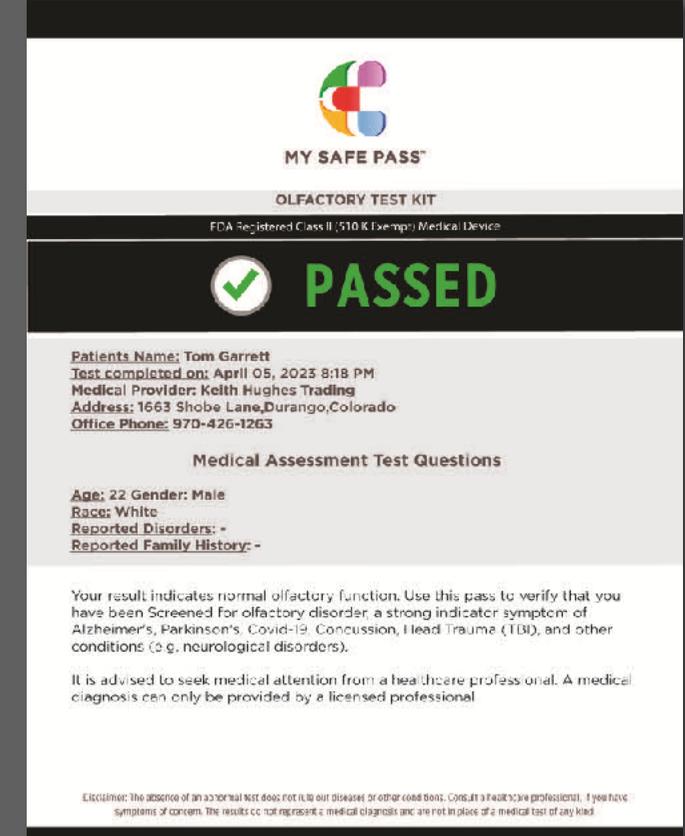


Auto email view

Administrator Log In

Administrators have secured access to all patients' history as well as additional data.

MSP provides each administrator with a secured username and password allowing direct access to all patients' data and history.

Start by logging in to your back-end portal.

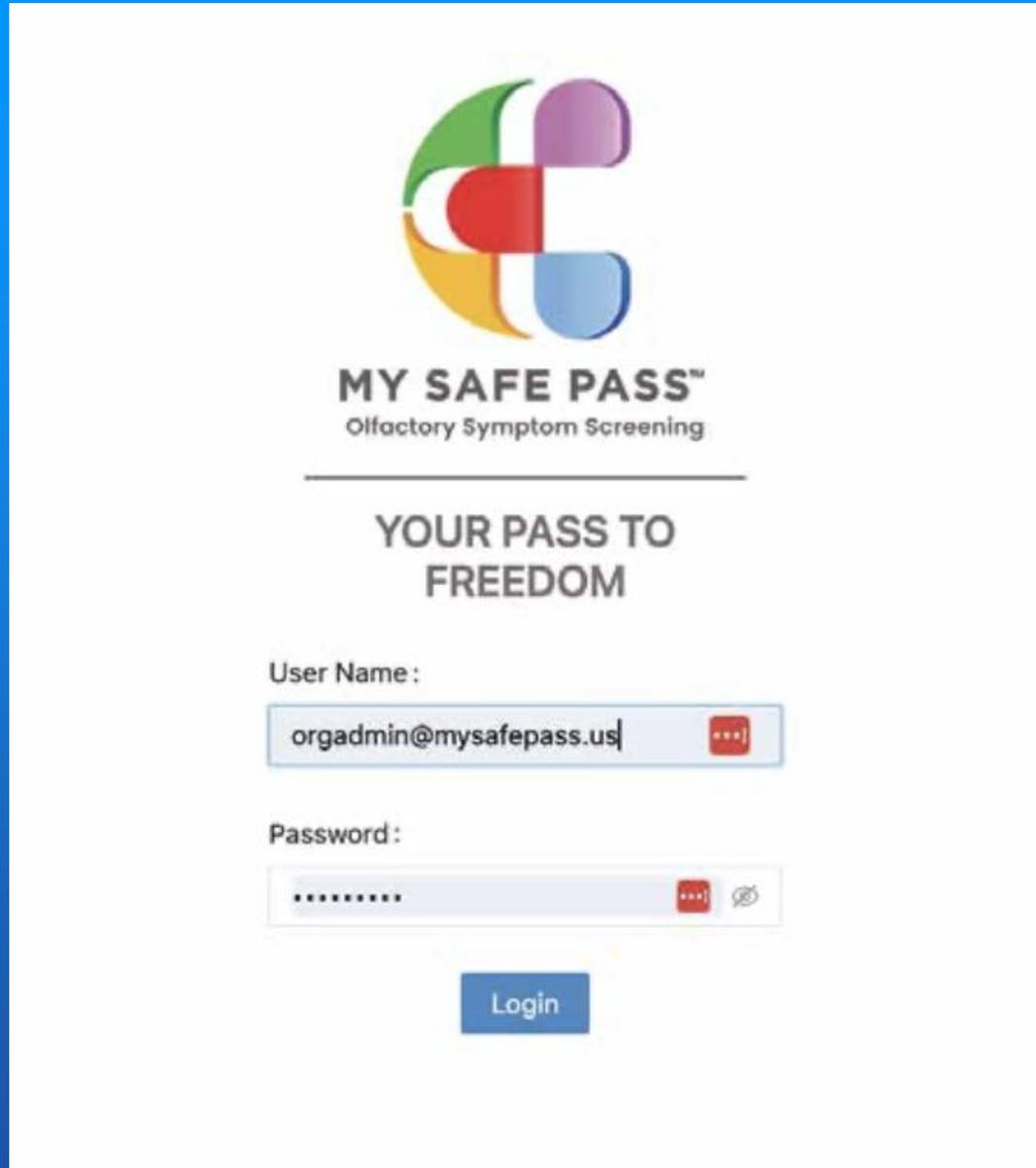

MY SAFE PASS™
Olfactory Symptom Screening

**YOUR PASS TO
FREEDOM**


User Name:

Password:

Login

Dashboard

Your dashboard will give you access to your users, location map data, geographical information via charts and graphs, and your pass/ fail data in numeric and graph format.

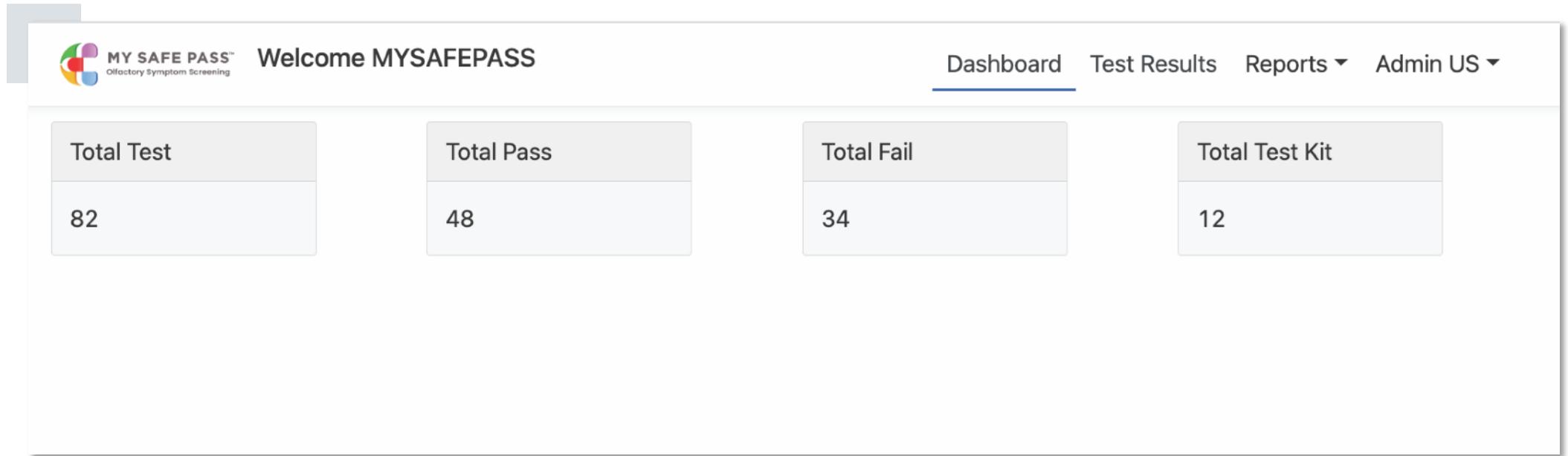

Location Map Reporting

The location map reporting screen gathers data using approximate locations and displays it in a map format. This allows you to monitor your organization by location to quickly recognize any potential symptom alerts. This screen will show you your passed and failed data by area and also by date.

The screenshot shows the MY SAFE PASS web application interface. At the top, there is a navigation bar with the logo, the text "Welcome MYSAFEPASS", and links for "Dashboard", "Test Results", "Reports", and "Admin US". Below the navigation bar, the page title is "Location Map". There are two radio button options: "Screening Performance of Current Year" (which is selected) and "Screening Performance by Date". The main content area features a map of the United States and surrounding regions, including Canada, Mexico, and parts of the Caribbean. The map displays data points as blue circles with numbers. One circle in the Northeast region contains the number "10", and another circle in the Southeast region contains the number "3". The map includes standard Google Maps controls like a zoom in (+) and zoom out (-) button, and a full-screen button. At the bottom of the map, there is a copyright notice: "© 2023 Copyright: MySafePass, All rights reserved".

Data Location


Using the data location view, you can easily convert your location data into numeric format. You can sort by country, state and city.

The screenshot shows the 'Data Location' report in the MY SAFE PASS system. The page header includes the logo, 'Welcome MYSAFEPASS', and navigation links for 'Dashboard', 'Test Results', 'Reports', and 'Admin US'. The report title is 'Data Location'. There are two radio button options: 'Screening Performance of Current Year' (selected) and 'Screening Performance by Date'. Below the options is a table with columns for Country, State, City, Passed, and Failed. The table contains three rows of data. At the bottom right of the table area, there is a pagination control showing '1-3 of 3 items' and '10 / page'. A copyright notice '© 2023 Copyright: MySafePass, All rights reserved' is located at the bottom of the page.


Country	State	City	Passed	Failed
NA	NA	NA	5	2
United States	South Carolina	Columbia	1	0
United States	South Carolina	Forest Acres	7	0

Pass/Fail Data

Access your passed/failed data screen to view an overview of your company's performance by date.

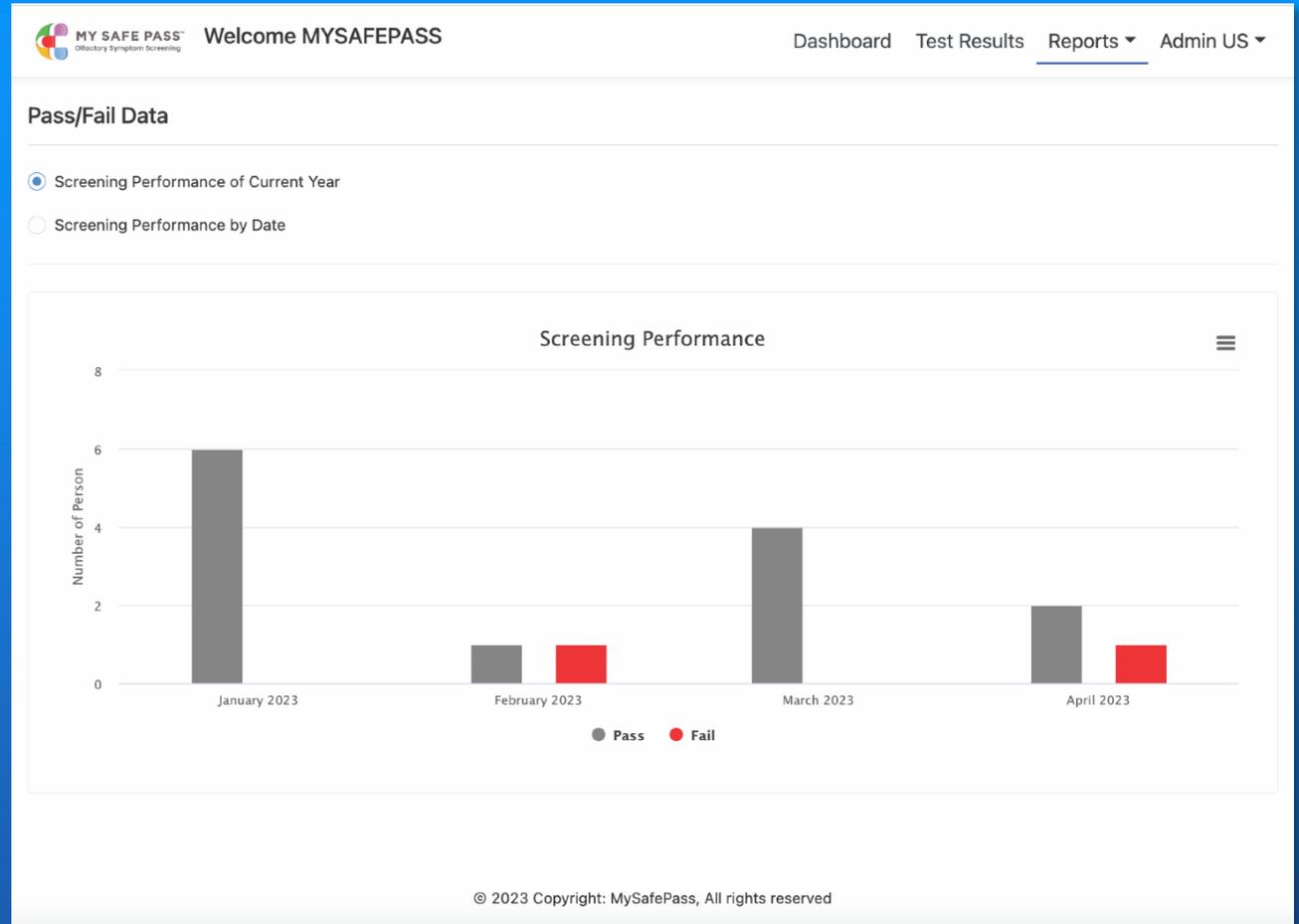

**Olfactory dysfunction can be due to other causes including neurodegeneration disease, head trauma or other viral infections and this is not to screen or diagnose a specific disease or condition.*

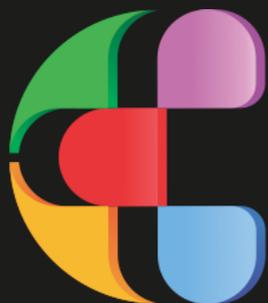

MY SAFE PASS™

Olfactory Test Kit

Symptom screening test for
Neurological Disorders, Concussion TBI, Viruses,
and other neurological diseases that cause olfactory
disorders.

www.mysafepass.us

www.patriotconnectionspe.com

843-457-0595

jeffrey@patriotconnectionspe.com